




Lifeloc FC Series

Quick Start Guide

Unlock the Power
of Alcohol Testing

Unit Basics

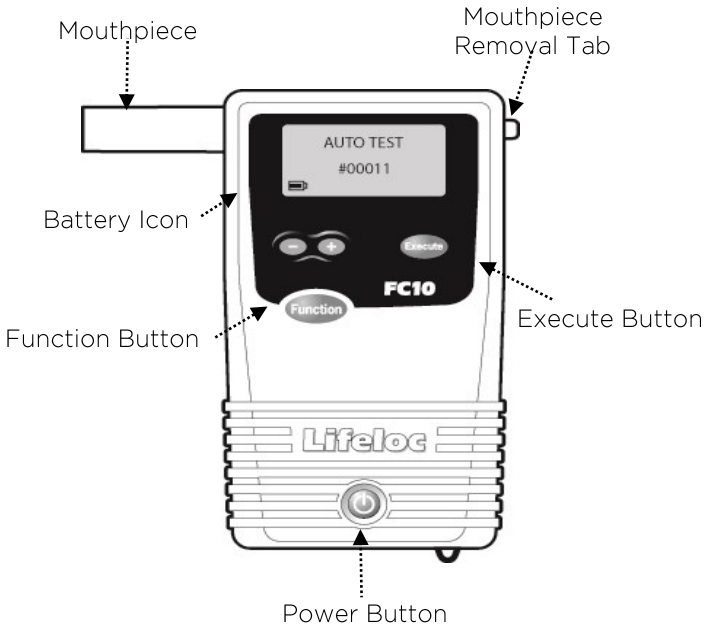


Attaching a Mouthpiece

Remove the mouthpiece from its wrapper, making sure not to touch the end which the subject will be blowing into.

Attach the mouthpiece to the port on the back of the FC unit.

Line up the mouthpiece port over the holes in the back of the mouthpiece. Press in place.

Ensure it is securely attached.

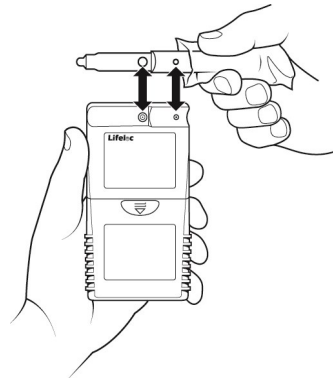


Prior to Taking a Test

IMPORTANT NOTES

Prior to test taking, you should observe the subject for 15 minutes, making sure the subject does not eat or drink anything during that time. This will prevent mouth alcohol from adversely affecting the test.


The measurement range is .000 to .600 BAC. If you receive the message ">0.6 BAC" after a test, please proceed with caution. This result is unusually high and the BAC result is above .600 BAC which is the maximum reading which can be displayed. The subject may require medical attention.

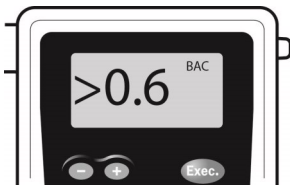

Taking a Test

Auto Test Mode - Easiest

- Proceed with the 15 minute observation period
- Press the **Power** button to start the unit
- Unit will run a self diagnostic check
- Verify the display reads "AUTO TEST"
- Attach a new mouthpiece to the back of the unit
- Have the subject blow steadily into the mouthpiece for about 4 seconds (listen for the beeping prompt to guide you)
- Read the Result

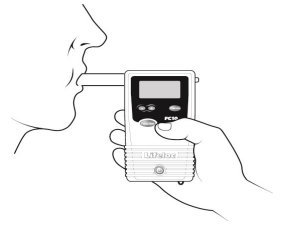


Manual Test Mode – Use when subject is unable to provide sufficient breath sample

- 15 minute Observation
- Power on unit
- Press **Function** until display reads "MANUAL TEST":
- Attach a new mouthpiece to the back of the unit
- Have subject blow steadily
- Near end of breath, press **Execute**
- Read the Result

Passive Test Mode – Detects alcohol only, readings are POS or NEG

- 15 minute Observation
- Power on unit
- Press **Function** until display reads "PASSIVE TEST"
- No Mouthpiece Needed
- Hold the back of the unit about 4" from the Subject's mouth
- Have the subject blow towards the Port (orange colored port)
- Press **Execute** while subject is blowing
- Read the Result

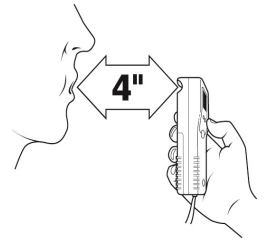

For **Open Container Testing**, follow the above Passive Test Mode steps while holding the unit over the container and press **Execute**.

Calibrations

Calibration of an FC sets your unit to a known alcohol concentration to enable accurate BAC (Breath Alcohol Concentration) results. You can use dry gas or wet bath solution. Lifeloc recommends you **Calibrate** your unit:

Once every **6 months**, regardless of how many tests you have performed;

Or more often if specified by your Internal Policies, Quality Assurance Plan, or Local Government Regulations.

Technical Support

For calibration or any other technical support questions, please contact your local Lifeloc distributor or Lifeloc Technical Support at support@lifeloc.com (1.720.317.2190).

Notice

- a) There must be an interval of at least 15 minutes after alcohol consumption prior to using this device as alcohol residue in the mouth may give a false reading. This includes mouthwash or similar products.
- b) A person's blood alcohol concentration (BAC) can continue to rise for up to 2 hours after drinking alcohol. Care should be taken if a caution result or a result is close to the designated BAC is indicated.
- c) It can take 10 hours or more for the alcohol level to return to a zero after a high blood alcohol level has been reached. A further test should be carried out later in the day of the following morning.
- d) For fastest testing times, store device at or near the operating temperature range of 32° - 130°F (0° - 55°C).
- e) DO NOT DRINK AND DRIVE. Lifeloc strongly recommends that no vehicle or machinery be operated after alcohol consumption. Even small quantities of alcohol can result in driving impairment.

For a complete list of features, settings and operation instructions please consult the device user manual.



12441 W 49th Ave, Suite 4
Wheat Ridge, CO 80033 USA
1.800.722.4872
www.lifeloc.com